
13. Mounting the attachment to the window handle "stop"

- Set the lever of the window handle horizontally
- Slide the receiving sleeve of the device onto the window handle.
- Place the prepared screw thread clamp 1 on the lever of the window handle as close as possible to its tip with the closure turned to the window and the screw 2 pointing upwards and tighten the screw 2.


14. Assembly of the stem from two parts

- Fit both parts of the handle (picture below) so that the markings A face each other and the holes C and D fit exactly together.
- Carefully press or hammer in the stem with holes B and D of the markings A turned upwards and on a not very hard surface (wood or similar).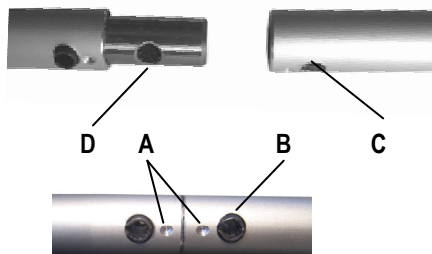

No more to air on table, chair, ladder
have to climb

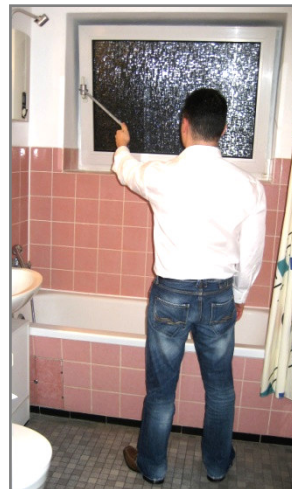Window opener window handle extension DESIGN Aids no. 02.40.04.5001

Instructions

Simple and universal solution
for hard to reach windows

Plotkina W-INVENTION

Adress: Flamweg 130, D-25335 Elmshorn
Tel.: +49 (0) 4121 7808948
Fax: +49 (0) 4121 7808949
E-Mail: info@w-invention.de
Shop: www.fenstergriff-verlaengerung.de

W-INVENTION

1. Purpose of the product / indication

The product is mainly intended for people who can not reach their windows for ventilation due to disability or illness.

Typical situations for use of the product: an office table, a bathtub, a kitchen worktop is placed in front of the window or the window handle is too high.

2. Existing application risks and contraindications


The manufacturer is not aware of any application risks.

The device is not a power amplifier. It can only be used by people who are able to operate an accessible window directly by hand on the window handle.


Persons with trembling or troubled hands should only use the devices permanently attached to the window handle to prevent unintentional damage to the window pane.

3. Konstruktion

The device has a handle 1, a handle (with or without padding) 2 and pivoting receiving sleeve 3, in which an anti-slip insert is installed on.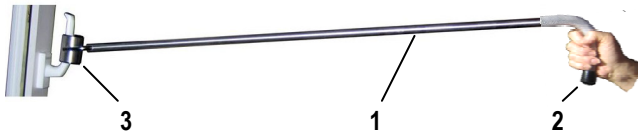

The stem of an over 115 cm long device can consist of two parts. The scope of delivery may include the device "stop" for permanent attachment of the device to a window handle and the "wall hook" for storing the device on a wall.

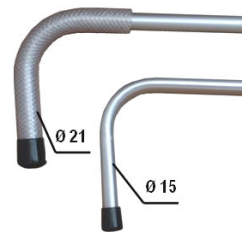With and without upholstery

Stop

Wall hook

4. Technical data / parameters

- Base length	about 95 cm
- Special lengths	to 230 cm
Weight:	
- Base length	about 0,3 kg
- Special lengths	to 0,6 kg
Receiving sleeve	with anti-slip insert
Opening of the receiving sleeve	about Ø 2,8 cm
Length of the receiving sleeve	about 5 cm
Handle without padding	Ø 1,5 cm
Handle with padding	Ø 2,1 cm
Attachment to window handle	Stop
Suspension on the wall	Wall hook

5. Delivery

Scope of delivery for orders according to a medical prescription in accordance with aid no. 02.40.04.5001 includes:

- window opener with padding *, Stop*, wall hook * and Instructions for use

* Otherwise, the extras upholstery, stop, wall hooks can be ordered as required.

6. Permissible operating conditions / locations

The device can be used on turn-only, tilt, turn-tilt windows and doors with L-shaped window handles, whose lever arm fits freely into the opening of the receiving sleeve of the device.

7. Application Notes

The device can be used either detachable for one or more windows or permanently attached to the window handle of a window.

The style of the device should not deviate more than 30 ° from its vertical position to the window when operating from the window. Otherwise you will either have to apply noticeably greater force or you will not be able to operate the window at all.

8. Cleaning Hints / disinfection instructions

The device can be cleaned with a soft cloth and soap with water.

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10. Warranty

3-year manufacturer's warranty on presentation of proof of purchase.

Incidentally, there are statutory warranty rights for the article.

9. Assembly and installation instructions

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